# Caya<sup>®</sup> diaphragm gel

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 Applikator:  
Hubert de Backer n.v.,  
9100 Sint-Niklaas, Belgium

# Caya<sup>®</sup> diaphragm gel

## Instructions for use (Please keep this leaflet!)

Caya<sup>®</sup> is a contraceptive gel for use with natural latex rubber or silicone diaphragms and cervical caps.

**Application**  
Before inserting into the vagina, place about one teaspoonful (about 4 ml) of Caya<sup>®</sup> Gel in the diaphragm or cervical cap. Then follow the instructions for inserting the diaphragm or cervical cap.

The cellulose gel creates an additional mechanical barrier between the diaphragm (or a cervical cap) and the uterine orifice. The lactic acid with its physiological pH of 3.8 has a motility inhibiting effect on the sperms – they are slowed in their forward movement or stopped.

**Composition**  
Water, lactic acid, sodium lactate, cellulose, sorbic acid

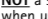
**Storage**  
Store in a dry place between 4 °C and 25 °C. Always keep the tube closed. For minimum shelf-life, see printing on tube.

 Use within 3 months of opening.

**Side effects**  
Caya<sup>®</sup> Gel, because of its pH, is well tolerated by the vaginal flora. It can be used for as long and as often as required. If any problems arise when using Caya<sup>®</sup> Gel, please stop using it and consult your doctor or pharmacist.

**Contraindications**  
There have been no reports of women experiencing side effects during the use of Caya<sup>®</sup> Gel.

**Important**  
Caya<sup>®</sup> Gel offers **NO** protection against sexually transmitted diseases. It is only effective when used together with a diaphragm (e.g. Caya<sup>®</sup>contoured diaphragm) or a cervical cap. Caya<sup>®</sup> Gel **IS NOT** a safe contraceptive method when used alone without a mechanical barrier. Caya<sup>®</sup> Gel is soluble in water and is compatible with natural latex rubber or polyurethane condoms.

 Applikator:  
Hubert de Backer n.v.,  
9100 Sint-Niklaas, Belgium

**Use of vaginal applicator**  
The vaginal applicator is used optionally if additional gel is to be introduced in front of the already inserted diaphragm if intercourse is repeated.

- Fill the applicator with Caya<sup>®</sup> Gel. Fit the applicator on the tube and carefully squeeze the gel out of the tube until the dosage mark is reached.
- Remove the applicator from the tube.
- Introduce the applicator, aperture first, as deeply as possible into the vagina.
- As soon as you have correctly positioned the applicator, empty it by pressing gently on the plunger.
- Remove the empty applicator from the vagina without pulling the plunger, since otherwise this would draw the gel back into the applicator barrel.
- This vaginal applicator is reusable. However, it should be washed in warm water after every use.

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KESSEL medintim GmbH