English

Please read this leaflet carefully before use.

YES VM

YES VM water-based organic Vaginal Moisturising gel uses a patented technology to provide natural, long-lasting moisture to dry vaginal mucosa.

YES VM is available in 100ml tubes and packs of pre-filled applicators.

- When menopausal estrogen depletion causes itching, burning and irritation of the dry vaginal tissues, YES VM gives effective, fast and long-lasting relief, and restores intimate comfort.
- YES VM uses bio-adhesive plant polymers which adhere to the vaginal walls and provide prolonged hydration for up to 3 days.
- Case studies show that YES VM is fast-acting and long-lasting while replenishing intimate moisture safely and naturally.

Provides symptomatic relief from:

- Vaginal dryness
- Menopausal Vaginal Atrophy
- Itching
- Irritation, burning and soreness
- Dyspareunia.

Temporary vaginal dryness can have various origins: after childbirth, when breastfeeding, and when taking oral contraceptives. Certain anti-depressants and anti-histamines can cause intimate dryness, as can extreme stress.

YES VM is:

- A soft, clear bio-adhesive gel
- Discreet no smell, taste, colour, stickiness or fallout. Close to the texture and feel of a woman's own lubrication
- pH matched to restore and stabilise the protective acidity of the intimate environment, supporting vaginal health
- Hormone-free
- Free from known mucosal irritants, and ingredients such as glycerine, petroleum products, parabens (hormone mimics) which can disrupt or compromise intimate health
- Gentle and natural, for even the most sensitive conditions
- Guaranteed purity through its certified organic status
- Safe for use with latex and polyisoprene condoms
- Trusted by Health Care Professionals and women to bring soothing, safe and welcome relief to the debilitating symptoms of menopausal dryness
- A non-hormonal natural alternative to synthetic chemical intimate moisturisers.

When cancer treatment (surgery or chemotherapy) has resulted in vaginal dryness, YES VM is safe for hormone sensitive conditions, and successfully moisturises delicate or traumatised intimate tissues. YES VM is also suitable for use with dilators, whether or not these are part of a cancer recovery programme.

Ingredients

Water, Flax extract*, Aloe vera*, Locust bean gum*, Guar gum*, Sodium chloride, Xanthan gum, Potassium sorbate, Citric acid, Phenoxyethanol.

* Organically grown ingredients.

95% organic.

Protected by International Patents.

Storage: Replace cap when not in use. Store in a cool dry place between 5+30°C. Use before expiry date. Keep out of direct sunlight.

Caution: This device is not a contraceptive or a spermicide.

Warning: If the seal on any tube enclosed is damaged or opened DO NOT use, return to place of purchase. If irritation or discomfort occurs, discontinue use and seek medical advice. YES VM is not compatible with Polyurethane condoms.

Why certified organic?

When you buy something that is certified organic, not only are you making the best choice for your own wellbeing, your purchase is also supporting a system that respects human health, animals, wildlife and the environment. Certified organic products meet stringent standards and organic certification is your assurance that our ingredients and products have been grown and processed according to strict procedures. Production, processing, distribution and sales are monitored to maintain the finest organic and ecological integrity.

Packaging and recycling

YES aims to minimise the environmental impacts of all our packaging. So please do help us achieve that aim. All our bottles are MDPE. Polyethylene (MDPE) is the most widely accepted plastic by recycling centres. The cartons, boxes and package inserts can all be recycled.

100% Guarantee: YES products are 100% satisfaction guaranteed. Please contact us by phone or email if you have any concerns or questions about this guarantee.

Certified Organic by: The Soil Association, Bristol, UK.

Approved by: The Vegetarian Society.

Manufactured in Great Britain for

The YES YES Company Ltd.

The Timber Barn, Selborne Road, Greatham, Liss, Hampshire, GU33 6HG, United Kingdom.

YES logos are registered trademarks of YES Syzygy Ltd.

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Websted: www.yesyesyes.org

E-mail: hello@yesyesyes.org

Directions for use - YES VM in pre-filled Applicators

When you want to reach higher in the vagina, this product offers you a discreet and convenient way to deliver lubrication and moisture in the right amount to the source of discomfort.

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This product is a soft clear gel, easy to dispense and distribute, and designed for finger application, which allows you to decide how much you want to put where. Simply squeeze the soft gel on to your hand and use your finger tips to apply the desired amount to the vaginal tissues. YES VM may also be used to moisturise the labia. (No applicator included)

YES VM - frequency of use

A 5ml application used every 2-3 days, is typically sufficient to maintain a moist, comfortable vagina.

Provides intimate moisture for up to three days in typical cases of vaginal dryness.

Cases of severe dryness will benefit from more frequent use (1 application every 1-2 days) initially until improvement is gained.

If using to alleviate painful sex, apply as required before intercourse and to ensure comfort afterwards.

Iso-tonic[®], the iso-tonic/iso-osmotic nature of YES VM is carefully matched to human tissues.

Why is this balance important? Products which are hypertonic (often containing chemicals such as glycerine and glycols) can cause irritation to intimate tissues.

By being iso-tonic/iso-osmotic[®], YES VM protects against such irritation.

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YES WB

Is a water-based, personal lubricant. This luxurious silky gel is non-sticky, odour and taste free and feels like natural lubrication.

It is pH matched to vaginal pH to help protect against bacterial and yeast infections.

Compatible with natural rubber latex and polyisoprene condoms.

YES OB

Is a plant-oil based, longer-lasting Lubricant and intense hypoallergenic moisturiser and massage oil, which is soothing and richly emollient to nourish delicate intimate tissues. YES BUT is iso-tonic and pH balanced for anal sex. YES BUT is compatible with natural rubber latex, polyisoprene condoms.

YES VM (Vaginal Moisturiser)

Is a long-lasting natural vaginal moisturising gel perfect for regular/daily use. It is designed to revitalise and hydrate to relieve vaginal dryness and discomfort. YES BUT is compatible with natural rubber latex and polyisoprene condoms.

YES BUT

Is a water-based rich and natural and sensuous anal lubricant gel which moisturises and lubricates intimate tissues. YES BUT is iso-tonic and pH balanced for anal sex. YES BUT is compatible with natural rubber latex, polyisoprene condoms.

YES OB VANILLA

Is a luxurious plant-oil based natural lubricant and massage oil delicately infused with organic Madagascan Vanilla, whose rich oils and butters nourish and condition skin, giving lasting lubrication and comfort.

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YES CLEANSE

Two natural intimate washes, one rose scented, one fragrance-free, which are sensitively formulated to respect the intimate ecology, with an effective and refreshing foam cleansing action. Each bottle contains over 200 applications.

YES DG (Double Glide)

This product enables a thrilling new dimension in love making, by layering water-based on top of oil-based to create a sublime and lasting glide. (YES DG comprises 1x 80ml tube of YES OB and 1x 100ml tube of YES WB).

YES INTRO


Is our easy way for you to try three different YES natural lubricants in three different formats, with a generous saving on price.

The YES range


Whether you use a little or a lot, for pleasure or for health, discover the full YES collection of natural and organic intimate products, in a range of sizes and variants to meet your differing needs.

YES OB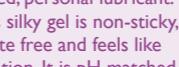

YES VM

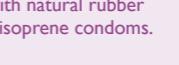YES BUT


YES OB VANILLA

YES CLEANSE

YES DG (Double Glide)

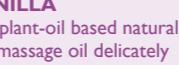YES INTRO